Operationsanleitung  
*Surgical Technique*

**INTEOS<sup>®</sup> Humerusplatte 3.5 proximal**  
***INTEOS<sup>®</sup> Humerus Plate 3.5 proximal***  
innovatives proximales Humerusplattensystem  
*innovative proximal Humerus Plating System*

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Achtung: Operationsanleitung beachten  
*Attention: Be aware of the manual of surgical technique*

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#### Abbreviations:

AO-QC	AO-Quick Coupling
Mini-QC	Mini-Quick Coupling
WS	Wrench Size
as	angle stable
nas	not angle stable
av	angle variable
st	self-tapping
mis	minimal invasive
-S	sterile
-T	Trial

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#### Warning

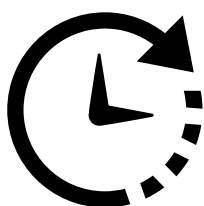
*This description does not suffice for immediate usage of the implants and instruments. A briefing by a surgeon experienced in this field in the handling of these instruments is highly recommended.*

<b>Vorwort</b> <i>Preface</i>	<b>UA-1</b>
<b>Einleitung</b> <i>Introduction</i>	<b>UA-2</b>
<b>Implantatspezifische Informationen</b> <i>Implant specific information</i>	<b>UA-3</b>
Indikationen <i>Indications</i>	UA-3.1
Kontraindikationen <i>Contraindications</i>	UA-3.2
Patientenlagerung und Zugänge <i>Patient positioning and approaches</i>	UA-3.3
Plattenspezifische Details <i>Plate specific details</i>	UA-3.4
Implantatspezifische Informationen <i>Implant specific information</i>	UA-3.5
<b>Lage der INTEOS® Humerus 3.5 prox.</b> <i>Position of the INTEOS® Humerus 3.5 prox.</i>	<b>UA-4</b>
<b>Zielbohrblock für INTEOS® Humerus 3.5 prox.</b> <i>Drill guide for INTEOS® Humerus 3.5 prox.</i>	<b>UA-5</b>
<b>Nachwort</b> <i>Epilogue</i>	<b>UA-6</b>

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*Hofer Medical Solutions gladly offers detailed training in safe handling and various surgical techniques.*

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Tel.: +43 (0)3382 53388  
Mail: office@hofer-medical.com

**Please contact our 24/7 service hotline:**

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*This document provides information about the handling of Hofer Medical Solutions implants and instruments.*

*This operation manual shall be considered as an addition and under no circumstances as a substitute to existing literature about surgical methods within orthopaedics and traumatology.*

*The content shall be regarded as a recommendation for a standardized procedure of how to apply the products without addressing the issues of any further necessary tasks, additional operative actions and possible extensions of the surgical technique.*

*The actual selection of the most suitable implant and its implantation method has to happen exclusively by the surgeon based on his education and the individual diagnostic findings.*

*All illustrations printed here have a purely symbolic character to support the description of the surgical technique and can vary.*

*These operation instructions don't contain any details on the use of the instruments. Corresponding documents are available in the form of*

- *Usage instructions for instruments: intra- and postoperative handling*
- *Usage instructions for implants (each implant is enclosed)*

*Please ensure that the diagnosis and determination of the treatment plan are left up to the surgeon.*

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*The INTEOS® Small Fragment System from Hofer GmbH & Co KG (HOFER) is a multidirectional and fixed-angle Small Fragment Implant System based on the fixative internal concept.*

*This system generally serves to treat diaphysial, metaphysial and epiphysial fractures in small bones.*

*All Hofer Medical Solutions products are the result of collective development by experienced users and our own engineers. This successful collaboration leads to products that fulfil the anatomical and functional requirements of the relevant structure, from the easiest fractures to comminute fractures, on the basis of the anatomically shaped, low-profile designs as well as the various supply possibilities.*

**General advantages of the system:**

- *Minimised irritation of ligaments and soft tissue as a result of flat plate and screw geometry*
- *Minimally protruding screw heads*
- *High angle flexibility of up to 50° when fully locked*
- *Very versatile as a result of the seven-plated structure (various holes for optimised screw placement) on relevant plates*
- *Angular stability in shaft area: maximised security in cases of bone instability*
- *Advantageous system as screws and instruments are compatible with various other plates from our HS3.0 range*
- *3-point support possible (conversion of bending stress into push and pull tensions) in larger, extendable bone defect areas*

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## INTEOS® Humerus 3.5 proximal

### UA-3.1 Indication

- *Displaced two-, three- and four-fragment fractures of the proximal humerus, as well as of osteoporotic bones, on the condition of a stable fixing in the distal (shaft) fragment*
- *Pseudoarthrosis in the proximal humerus with support measures for secure fusion and fixing*
- *Osteotomies on the proximal humerus*

### UA-3.2 Contraindication

- *Children who are still growing*
- *Infection or inflammation (acute, chronic, local)*
- *Poor circulation in the affected area*
- *Unsuitable implant site*
- *Possible or pre-existing sensitivity to the material*
- *Patients with little or no compliance in relation to the observance of post-operative rehabilitation recommendations*
- *Further information about patient selection is available in the implant manual*

### UA-3.3 Patient positioning and approaches

#### *Patient positioning:*

- *The patient will normally be operated on in the Beach Chair or supine position*

#### *Approaches:*

- *The deltopectoral or transdeltoid approach has been proven*

#### *Tip:*

- *For transdeltoid approaches, it is extremely important to avoid any impingement of the nearby axillary nerve near the ovals and the 3 plate holes above*

### UA-3.4 Plate and screw specific details

- *Anatomically shaped plate for right and left*
- *Orientation of the plate position nearby the sulcus bicipitalis for easy re-alignment of the fracture*
- *Pre-angled screw position with guided insertion device and optimised screw positioning*
- *Screw pairs crossed over one another ensure simultaneous higher resistance against misplacement of the head fragment and minimized risk of the screw moving into the joint as well as better addressability of fractures in the ventral and dorsal areas*
- *Possibility of individual screw placement without the help of the guided insertion device but still with high angle flexibility*
- *Slimmer distal end of the plate for lower alteration of the deltoid muscle*
- *Extremely high level of stability against bending*
- *Rounded screw head*
- *Non-invasive screw tip*
- *„Stick Fit“ - the stable connection between screw driver and screw*
- *ForceDrive 10*

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*Screw diameter 3.5 mm  
 Core diameter 2.4 mm  
 Head diameter 5.6 mm  
 Drill bit for threaded hole angle stable & polyaxial 2.7 mm  
 Compatible to ForceDRIVE T10*

Titan	Durchmesser	Länge	VPe
<i>Titanium</i>	<i>Diameter</i>	<i>Length</i>	<i>PU</i>
716-110-035-010	Ø 3,5 mm	10 mm	1
716-110-035-012	Ø 3,5 mm	12 mm	1
716-110-035-014	Ø 3,5 mm	14 mm	1
716-110-035-016	Ø 3,5 mm	16 mm	1
716-110-035-018	Ø 3,5 mm	18 mm	1
716-110-035-020	Ø 3,5 mm	20 mm	1
716-110-035-022	Ø 3,5 mm	22 mm	1
716-110-035-024	Ø 3,5 mm	24 mm	1
716-110-035-026	Ø 3,5 mm	26 mm	1
716-110-035-028	Ø 3,5 mm	28 mm	1
716-110-035-030	Ø 3,5 mm	30 mm	1
716-110-035-032	Ø 3,5 mm	32 mm	1
716-110-035-034	Ø 3,5 mm	34 mm	1
716-110-035-036	Ø 3,5 mm	36 mm	1
716-110-035-038	Ø 3,5 mm	38 mm	1
716-110-035-040	Ø 3,5 mm	40 mm	1
716-110-035-042	Ø 3,5 mm	42 mm	1
716-110-035-044	Ø 3,5 mm	44 mm	1
716-110-035-046	Ø 3,5 mm	46 mm	1
716-110-035-048	Ø 3,5 mm	48 mm	1
716-110-035-050	Ø 3,5 mm	50 mm	1
716-110-035-052	Ø 3,5 mm	52 mm	1
716-110-035-054	Ø 3,5 mm	54 mm	1
716-110-035-056	Ø 3,5 mm	56 mm	1
716-110-035-058	Ø 3,5 mm	58 mm	1
716-110-035-060	Ø 3,5 mm	60 mm	1
716-110-035-062	Ø 3,5 mm	62 mm	1
716-110-035-064	Ø 3,5 mm	64 mm	1
716-110-035-066	Ø 3,5 mm	66 mm	1
716-110-035-068	Ø 3,5 mm	68 mm	1
716-110-035-070	Ø 3,5 mm	70 mm	1
716-110-035-072	Ø 3,5 mm	72 mm	1
716-110-035-074	Ø 3,5 mm	74 mm	1
716-110-035-076	Ø 3,5 mm	76 mm	1
716-110-035-078	Ø 3,5 mm	78 mm	1

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Screw diameter 3.5 mm  
 Core diameter 2.4 mm  
 Head diameter 5.6 mm  
 Drill bit for threaded hole 2.7 mm  
 Drill bit for cannulation 3.5 mm  
 not angle stable  
 Compatible to ForceDRIVE T10

Titan	Durchmesser	Länge	VPe
<i>Titanium</i>	<i>Diameter</i>	<i>Length</i>	<i>PU</i>
716-115-035-010	Ø 3,5 mm	10 mm	1
716-115-035-012	Ø 3,5 mm	12 mm	1
716-115-035-014	Ø 3,5 mm	14 mm	1
716-115-035-016	Ø 3,5 mm	16 mm	1
716-115-035-018	Ø 3,5 mm	18 mm	1
716-115-035-020	Ø 3,5 mm	20 mm	1
716-115-035-022	Ø 3,5 mm	22 mm	1
716-115-035-024	Ø 3,5 mm	24 mm	1
716-115-035-026	Ø 3,5 mm	26 mm	1
716-115-035-028	Ø 3,5 mm	28 mm	1
716-115-035-030	Ø 3,5 mm	30 mm	1
716-115-035-032	Ø 3,5 mm	32 mm	1
716-115-035-034	Ø 3,5 mm	34 mm	1
716-115-035-036	Ø 3,5 mm	36 mm	1
716-115-035-038	Ø 3,5 mm	38 mm	1
716-115-035-040	Ø 3,5 mm	40 mm	1
716-115-035-042	Ø 3,5 mm	42 mm	1
716-115-035-044	Ø 3,5 mm	44 mm	1
716-115-035-046	Ø 3,5 mm	46 mm	1
716-115-035-048	Ø 3,5 mm	48 mm	1
716-115-035-050	Ø 3,5 mm	50 mm	1
716-115-035-052	Ø 3,5 mm	52 mm	1
716-115-035-054	Ø 3,5 mm	54 mm	1
716-115-035-056	Ø 3,5 mm	56 mm	1
716-115-035-058	Ø 3,5 mm	58 mm	1
716-115-035-060	Ø 3,5 mm	60 mm	1

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### General instructions

- *The implant is anatomically shaped, however, should any adjustment be necessary, such adjustments can be made with the aid of the appropriate bending pliers*
- *The screws in the shaft area of the bone should always be locked bicortically*
- *The implantation of the INTEOS® Humerus 3.5 prox. takes place near the sulcus bicipitalis*

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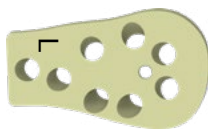
**General instructions**

- The guided insertion device is positioned correctly on the plate by lateral guide pins
- Notice must be taken of the "L" and "R" labels
- The correct drill sleeve must be used (exact fit of the drill sleeve in the holes (4) must be ensured)

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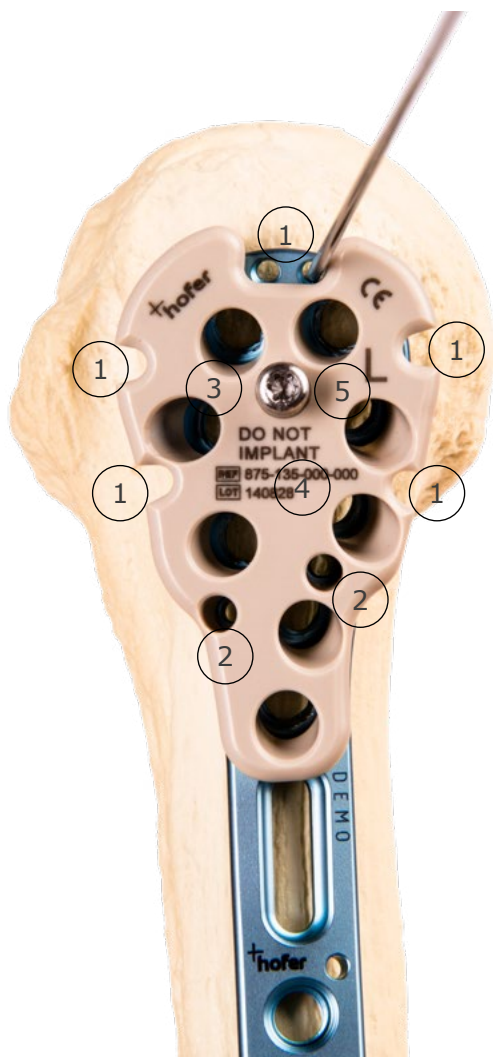
1. Use of Kirschner's wires as a temporary means of attaching the plate or as an approach to the posterior drill hole, when including suture material
2. Use of Kirschner's wires as a temporary means of attaching the plate
3. Location hole of the drill sleeve to the guided and angled pre-drilling for the cap screw
4. Labelling of the guided insertion device
5. Sides labelled on the guided insertion device (L or R). Must fit with the INTEOS® Humerus 3.5 prox. plate

**Bohrhilfe für INTEOS® Humerus prox 3.5**  
 Drill Guides for INTEOS Humerus prox 3.5

Stahl Steel	Für Bohrer For Drill Bit	Länge Length	VPe PU
<b>Handgriff für Schraubbohrhülse / Handle for screwable drill guide tip</b>			
811-956-000-125		125 mm	1
<b>Schraubbohrhülse / Screwable Drill Guide Tip</b>			
811-956-027-060	Ø 2,7	60 mm	1
811-956-027-080	Ø 2,7	80 mm	1

**Bohrhülse für Zielbohrblock INTEOS® Humerus prox 3.5**  
 Drill Sleeve for Aiming Device INTEOS Humerus prox 3.5

Stahl Steel	Für Bohrer For Drill Bit	Länge Length	VPe PU
875-135-027-060	Ø 2,7 mm	60 mm	1
875-135-027-080	Ø 2,7 mm	80 mm	1

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© 2020 Hofer GmbH & Co KG. All rights reserved.

*Operation instruction booklets, handbooks, information brochures and software are protected by copyright. Copying, duplication, translation or conversion of said document through any electronic means or machine readable form, whether that be in full or in part, is not permitted without prior written authorisation from Hofer Medical Solutions.*

*All further rights regarding the software are defined in the provided licensing terms.*

*The Hofer Medical Solutions logo is a trademark of HOFER GmbH & Co KG and is registered in Austria and additional countries.*

*The rights to other brand and product names mentioned in this scripture are reserved by their proprietors and are accepted and recognised here. The naming of products that are not Hofer Medical Solutions products serves solely as a source of information and not of advertising. Hofer Medical Solutions accepts no liability regarding the selection, performance or efficiency of said products.*

*The methods and practices, etc. quoted in this scripture have been carefully tested and verified. However, should errors or suggestions of improvement be apparent, despite said verification, we politely ask you to disclose them to us. The surgical method portrayed here merely presents one possible method for the indication that is to be treated. It remains the responsibility of the relevant surgeon to either keep to the aforementioned osteosynthetic method or to vary the method according to individual needs. Direct and indirect complications as well as sequential complications are thus, in every case, excluded.*

*Should you have any further questions, please do not hesitate to contact us at any time.*

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Ihr Hofer Medical Solutions Team

*For us, a partnership is the result of a long and happy relationship in all areas of our work. Reliable bone healing for patients, optimal handling of the systems for surgeons and surgical personnel, as well as the simple preparation of the instruments, constantly strengthen this partnership.*

*For us, "creating" means more than just finding solutions in the form of innovative products.*

*It is the result of high standards, constant development, innovative products and excellent service for patients, surgeons and surgical personnel around the clock.*

*Please do not hesitate to contact us if you have questions about our company, our employees or our production methods.*

*Your Hofer Medical Solutions Team*